

## Pregnancy Test Strip



Questions? Call: 1-858-333-1122

PLEASE READ ALL THE INFORMATION IN THIS INSERT BEFORE PERFORMING THE TEST.

### INTENDED USE


Pregnancy Test Strip is a rapid pregnancy test. It detects the presence of the pregnancy hormone hCG (human chorionic gonadotropin) in urine. It is intended for home users and for prescription use.

### CONTENT OF THE TEST KIT


- One pouch containing a test strip and a desiccant.
- Urine collection cup.
- An instruction sheet.

### HOW TO USE TEST

1. Collect urine in the cup provided.
2. Remove the test strip from the sealed pouch and use it as soon as possible.
3. Immerse the strip into the urine with arrow pointing downward for 5 seconds. Urine level should not exceed the MAX line on the strip.
4. Take the strip out and lay the strip flat.
5. Read the test result at 5 minutes.  
Results read after 6 minutes may be inaccurate.

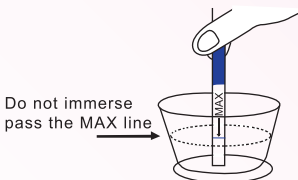

### HOW TO READ THE RESULT

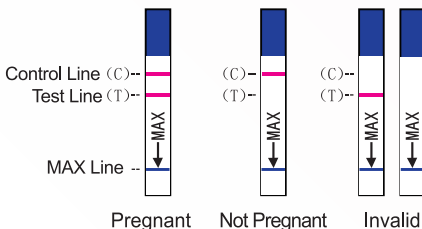

**Pregnant:** One pink line appears at the "C" region and one pink line appears at the "T" region.

**Not Pregnant:** One pink line appears at the "C" region and NO visible pink line appears at the "T" region.

**Invalid:** NO visible pink line appears at the "C" region with or without a pink line appearing at the "T" region.

### STORAGE

Store at 39°F-86°F (4°C-30°C) in the sealed pouch up to the expiration date. Open the pouch only shortly before the test.

## PRECAUTIONS

- Do not use the test kit if the pouch is damaged.
- Do not use after the expiration date.
- For in-vitro diagnostic use only. Not to be taken internally.
- The procedures should be followed precisely for accurate results.
- Single-use device. Do not reuse.
- Do not freeze.

## LIMITATIONS

- Medical treatment containing hCG can affect the test result.
- Certain health conditions may cause elevated hCG concentration. Always confirm your pregnancy with a physician.
- This device cannot be used to diagnose abnormal pregnancy.

## QUESTION AND ANSWERS

**Q: How does the Pregnancy Test Strip work?**

**A:** The test is designed to detect hCG in urine. hCG is produced by the human body during pregnancy.

**Q: When should I use the Pregnancy Test Strip?**

**A:** You can use the test as soon as the first day of the missed period.

**Q: How accurate is the test?**

**A:** The test can detect hCG in urine. However, some pregnant women may not have detectable amounts of pregnancy hormone hCG in their urine on the first day of the expected period or may have miscalculated the first day of their period.

**Q: What do I do if the result is positive (Pregnant)?**

**A:** You should go to your doctor to confirm that you are pregnant and get your doctor's professional advice on what to do next.

**Q: What do I do if the result is negative (Not pregnant)?**

**A:** You may not be pregnant. However, a false negative result (negative when pregnancy exists) may occur if the urine is too dilute or with a very early stage of pregnancy. If pregnancy is still suspected and you already missed your period, another urine specimen should be collected 2-3 days later and tested.

**Q: Do I have to use first-morning urine for testing?**

**A:** You can use the test at any time of day. However, in general, your first-morning urine contains the highest concentration of the pregnancy hormone hCG.

**Q: What do I do if the result is Invalid?**

**A:** If you have waited for 5 minutes and the result is invalid (no line appears at the "C" region), you should retest using a new test kit.

Manufactured for CorDx, Inc.  
9540 Waples St Unit C, San Diego,  
CA 92121, USA

PI-HCG-01-01, Rev. 05/05/2022